

SECURE® PADS®-2 ALARM

SAFETY SOLUTIONS
Your safety is our business®

WITH NURSE CALL INTERFACE

Can Be Used with Bed Pad, Chair Pad and Seat Belt

Model # PADS-2 • Made in PRC

NEW!
Easy-to-Use
Reset Button

FINALLY, A TAMPER-RESISTANT FALL MONITORING ALARM!

OPERATING INSTRUCTIONS:

1. Remove tamper-resistant screw on battery compartment cover on front of alarm. Remove battery cover. Insert three “AAA” batteries included in package.


PLEASE NOTE: Alarm will immediately sound once batteries are installed.

To shut alarm sound off press the Test/Reset button 3 times within 1.5 seconds.

Alarm will immediately stop sounding.

Slide battery compartment cover back in place. Tighten tamper-resistant screw. Make certain cover is securely in place.

2. Loosen locking security screw located on the side of alarm and slide the tamper-proof cover off the alarm. You will see the multiple settings and jacks as shown in Illustration A.
3. To begin making your setting decisions leave all four setting switches in the down or “ON” position. **Note: Leave Switch 1 – In Room Sound Switch in the down or “ON” position when testing the other 3 settings. Use the end of a pen to move the switches marked 2-Volume, 3-Tempo and 4-Tone.**
 - a. Switch 2 –Volume Switch. Move Volume Switch to desired decibel level. The decibel level range is from a high of 90 decibels with the switch in the down position to a low of 70 decibels with the switch in the up position.
 - b. Switch 3 – Tempo Switch. Move the Tempo Switch to desired tempo. The tempo range is from a quick tempo in the down position to a slow tempo in the up position.
 - c. Switch 4 – Tone Switch. Move the Tone Switch to desired tone. The tone range is a low pitched alarm sound in the down position to a high pitched alarm sound in the up position.
4. Slide connecting wire of pad through open slot at end of alarm and plug into opening marked “Pad Jack.” Tuck

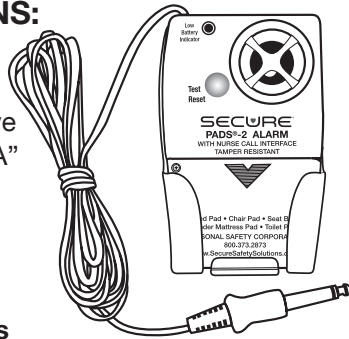

Nurse Call Cable Sold Separately

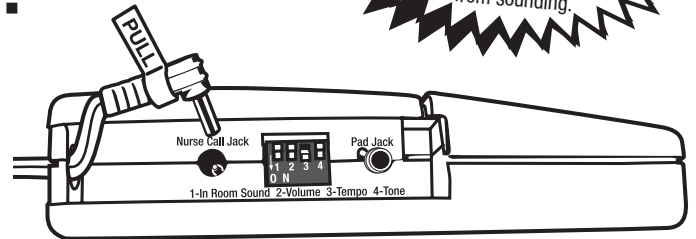


Illustration A
New & Improved! Now with In-Room On/Off Switch!

“Pull” tabs into compartment. Slide nurse call and pad connecting wires up through open slot at end of alarm. To remove jacks pull up on tabs marked “Pull.” **CAUTION! ALWAYS remove sensor pad pin from alarm by pulling straight up on the white “PULL” tab. DO NOT attempt to remove pin any other way or you may damage pin and void warranty.**

5. If using a nurse call cable determine if you want to silence the alarm in the resident’s room. If you want the alarm to sound ONLY at the nurse’s station and DO NOT want the alarm to sound in the resident’s room when using the nurse call feature, move the switch marked 1-In Room Sound up to the “OFF” position. If you want the alarm to sound in BOTH the room and at the nurse’s station slide the switch down to the “ON” position.

PLEASE NOTE: IF YOU DECIDE TO SILENCE THE ALARM IN THE RESIDENT’S ROOM WHEN USING THE NURSE CALL SYSTEM, YOU MUST SLIDE THE 1- IN ROOM SOUND SWITCH UP TO THE “OFF” POSITION. IF YOU WANT THE ALARM TO SOUND IN THE RESIDENT’S ROOM THE NEXT TIME USED YOU MUST MOVE THE IN ROOM SOUND SWITCH DOWN TO THE “ON” POSITION.

6. Place alarm in holder included. Press alarm’s back firmly against holder to secure Velcro® pieces together. When it is necessary to remove the alarm from holder simply pull up on the alarm to separate the Velcro® pieces.
7. Affix the alarm and holder securely to the side of the bed or chair out of sight and reach of the resident or person under care. Simply wrap the Velcro® strap with the rubberized non-slip backing attached to the back

NOTE:
For resident and patient safety & to make this alarm as tamper-resistant as possible you must press and release the TEST/RESET button 3 times within 1.5 seconds to stop the alarm from sounding.

PADS[®]-2 ALARM

WITH NURSE CALL INTERFACE

Can Be Used with Bed Pad, Chair Pad and Seat Belt
Continued

of the holder tightly around railing of bed or wheelchair.

CAUTION: Do not affix to PVC rail. The strap may permanently stain the rail. Holder placement should make alarm easily accessible to staff member or caregiver. Secure any excess monitoring pad cord to bed or wheelchair rail to avoid damaging cord.

8. If using the nurse call interface connect the standard 1/4" nurse call wall jack into wall outlet for your nurse call system. Check to make certain your nurse call system is functioning properly.
9. To test alarm for battery and proper functioning press the Test/Reset button. The alarm will sound 4 times to indicate that it is set to function properly. You can stop the alarm from sounding anytime by pressing the Test/Reset button 3 times within 1.5 seconds. When user returns to the pad the alarm will automatically reset.
10. Place resident so that resident's buttocks or shoulder are in center of bed or chair pad. When resident is placed

on the pad the alarm will automatically be set to alarm if resident attempts to get out of bed or chair. **To stop the alarm from sounding press and release the Test/Reset button 3 times within 1.5 seconds.** When resident returns to pad the alarm will automatically rest.

11. The red "Low Battery Indicator" light will come on when it is time to change the batteries.

PLEASE NOTE: THIS ALARM SYSTEM MUST BE TESTED EACH TIME IT IS USED.

WARRANTY: Personal Safety Corporation warrants this product to be free from factory defects in materials and workmanship for a period of one year from the date of purchase. Warranty is void if this product is misused, abused or damaged. No warranty or responsibility of use expressed or implied.

CAUTION!


This product is not a restraint. It will not prevent a resident from getting out of their bed or chair. This product is designed as a monitor, alerting the staff when the resident has risen from their bed or chair. This product is to be used in conjunction with a total fall prevention program.

Other SECURE High Quality Fall Management Products


SECURE Waterproof Beveled Edge Safety Floor Mat (SBEV-1W)




SECURE One Year Seat Belt Set (SB-1 Set)

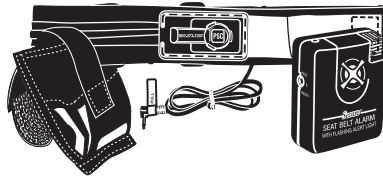

SECURE Floor Mat Alarm System (MAT-3 Set)

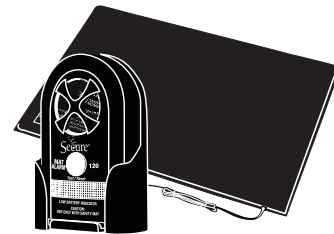

SECURE Magnet Alarm w/Protective Holder (MAG-3)

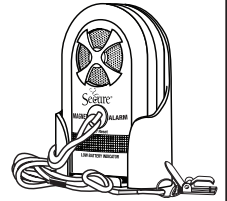

Please see our complete line of Secure[®] fall/wandering prevention & patient safety products at www.SecureSafetySolutions.com

Highest Quality...Lowest Prices...Guaranteed!

Personal Safety Corporation • Hiawatha, Iowa 52233 • 800.373.2873