



The Secure[®] Comfort Heel Pillow protects heels and ankles from skin irritation and friction burns, and helps prevent and heal decubitus ulcers. Unlike other similar heel pillows, the Secure[®] Comfort Heel Pillow does not have a stitched seam on the heel side of the pillow eliminating the benefits of the inner polyester fiber padding. The heel side of the pillow is fully padded to provide maximum user comfort.

The Heel Pillows are filled with polyester fiber, covered with a cotton fabric, and have a padded adjustable strap with a hook and loop closure for a custom fit. The Heel Pillows are machine washable and sized for most applications. One pair per package. One size fits most.

Recommended for patients at low risk for skin breakdown and friction burns.

MONITORING

Be sure to follow your facility's policies and guidelines for frequency of patient monitoring. Personal Safety Corporation recommends that this product be removed at least every two (2) hours to check for skin integrity, proper circulation and range of motion.

INSPECTION

Inspect before each use: check for broken stitches; or torn, cut or frayed material. DO NOT use soiled or damaged products.

APPLICATION INSTRUCTIONS

1. Unfasten the hook and loop padded strap.
2. Place the heel cushion on the foot so the stitched seam is visible along the sole, not up the leg, and the padded strap can be secured across the ankle.
3. Fasten the hook and loop padded strap for the desired fit.

CAUTION!

DO NOT allow the patient to walk or ambulate while wearing this product.

STORAGE AND HANDLING

This device is designed for use in normal indoor environments. This device may be stored in ambient warehouse temperatures at normal humidity levels. Avoid excess moisture or high humidity that may damage product materials.

WARRANTY: Personal Safety Corporation warrants this product to be free from factory defects in materials and workmanship for a period of 90 days from the date of purchase. Warranty is void if this product is misused, abused or damaged. No warranty or responsibility of use expressed or implied.

Please see our complete line of Secure[®] fall & wound care management products at www.PadAlarm.com.



Secure[®] Convex Pommel Cushion



Secure[®] Gel Cushion



Secure[®] Deluxe Arm Support



Please see our complete line of Secure[®] fall/wandering prevention & patient safety products at www.SecureSafetySolutions.com

Highest Quality...Lowest Prices...Guaranteed!

Personal Safety Corporation • Hiawatha, Iowa 52233 • 800.373.2873