

WATERPROOF BEVELED EDGE SAFETY MAT

With Glow In The Dark Edges

38" x 72" x 1" • Model # - SBEV-1W

The Secure[®] Beveled Edge Safety Mat has been biomechanically tested to reduce the risk of serious head injuries to .4% from an 18" sideways fall out of bed. See www.padalarm.com/beveledfloormat.htm for complete test results. The Secure[®] Beveled Edge Safety Mat has been designed to be used as part of a total fall management program. Please note that no safety mat can prevent all injuries as a result of an accidental rolling out of bed. However, this mat has been designed with a one-inch thick Ethylene Vinyl Acetate (EVA) foam for maximum fall impact absorption. In addition, the low one-inch profile makes it easy for residents to step on and off the mat. The beveled edges will minimize a shuffling resident's potential to trip when walking up to and onto the mat.

The beveled edges of the mat were also designed to easily roll a wheelchair onto the mat to safely transfer a resident from the chair to the bed or visa versa. Staff can also roll feeding or medical supply tables onto the mat for easy access to the resident. The mat will greatly assist in optimizing the potential to prevent serious injuries as a result of an accidental rolling out of bed.

UNIQUE FEATURES AND BENEFITS

- UNIQUE FEATURES AND BENEFITS: Waterproof cover to protect against all liquid spills (milk, juice, urine, etc.)
- Anti-Microbial Cover
- With Glow In The Dark Edges For Night Time Visibility
- Slip-Resistant On Both Sides
- Latex-Free
- Mildew-Resistant
- Flame Retardant
- One-Inch High-Impact Absorption EVA Foam
- Tri-Fold With Velcro Strap Closure For Easy Storage

INSTRUCTIONS FOR USE

1. Unfasten the Velcro[®] closure and open up the three sections of the mat. Make sure the "THIS SIDE DOWN" is facing the floor. Make note in the resident's care plan that a

safety mat is being used. Staff members, the resident, a roommate or any visitors must exercise caution when stepping on or off the mat to avoid tripping.



2. The Secure[®] Beveled Edge Safety Mat is 38" wide x 72" long, which is an ideal size for maximum protection against injuries as a result of accidentally rolling out of bed. Position the safety mat lengthwise along the side of the bed with the non-beveled edge of the mat positioned along the length of the bed. Make certain the lengthwise non-beveled edge of the safety mat closest to the length of the bed is about 1" under the bed. This will insure that the user will roll directly onto the safety mat in the event of an accidental fall. The three beveled edges of the mat will be facing out from the bed allowing a wheelchair or cart to easily roll onto the mat.

Note: If the mat is used with a low bed do not place the mat under the bed. Leave enough space so that the handrail will not damage the mat.

3. To store the mat simply fold one section onto the middle section and then fold the remaining section over the top and fasten the Velcro[®] closure.

CLEANING INSTRUCTIONS

The Secure[®] Beveled Edge Safety Mat is designed with a black waterproof cover to minimize the need for frequent cleaning. When necessary clean the mat with the same cleaning agents used to clean other vinyl materials used in your facility.

ONE YEAR LIMITED WARRANTY

Personal Safety Corporation warrants this product to be free from factory defects and workmanship for a period of one year from date of purchase. Void if this product's outer covering is misused, abused, cut or punctured. No warranty or responsibility of use expressed or implied.

WARNING!

There is no guarantee the user will be protected against all injuries as a result of a fall. Please properly position the safety mat according to the instructions above with each use to provide maximum protection against serious injuries as a result of a fall.



Please see our complete line of Secure[®] fall/wandering prevention & patient safety products at www.SecureSafetySolutions.com

Highest Quality...Lowest Prices...Guaranteed!

Personal Safety Corporation • Hiawatha, Iowa 52233 • 800.373.2873