


QUICK-RELEASE ADJUSTABLE WHEELCHAIR SEAT BELT

**w/ Quick-Release Push Button Buckle - Adjustable from 42"-62"
For Bariatric Patients Use Our 12" Seat Belt Extender (SBE-12)**

Model # SBNA-1 • Made in PRC

THE WHEELCHAIR SEAT BELT WAS DESIGNED TO BE PART OF A TOTAL FALL AND WANDERING MANAGEMENT PROGRAM. THE SEAT BELT WILL NOT PREVENT FALLS OR WANDERING. THE USE OF THIS PRODUCT MAY NOT BE SUITABLE FOR HIGH RISK RESIDENTS.

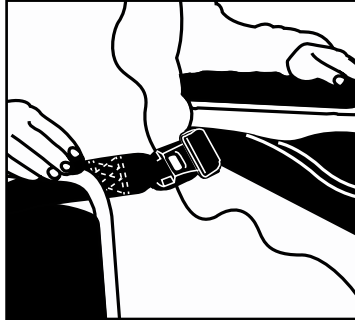

SEAT BELT WHEELCHAIR INSTALLATION

NOTE: If seat belt is being used on larger resident add Seat Belt Extender (SBE-12) (sold separately) to one or both ends of seat belt depending on need. Use clip on end of extender to attach to grommet at end of seat belt.

1. Unscrew bottom screw on each side of the seat back on back side of wheelchair.
2. Place seat belt on seat of wheelchair. Slide right and left ends of seat belt strap between wheelchair seat back and frame. Determine which grommet to use on the left and right sides of seat belt for approximate desired length for user. Insert screws from wheelchair through selected grommets of seat belt straps on both sides and attach seat belt to wheelchair. The seat belt can further be adjusted for patient or resident comfort by tightening or loosening the strap on the right side using the adjustable plastic piece if necessary.

TO USE WITH RESIDENT OR PATIENT

1. Open seat belt by pressing red button on buckle. Carefully assist user into chair. Insert metal end of seat belt into the red button buckle. Check to ensure buckle is fastened securely.

2. Insert hand between belt and patient or resident to ensure the belt is not too tight against user. Confirm with user that the belt is not too tight and make any further adjustments accordingly.
3. Clearly demonstrate to patient or resident how to unbuckle the belt by pressing the red button on the quick-release buckle. Refasten seat belt buckle.
4. Ask patient or resident if (s)he knows how to unbuckle the belt. If the answer is "No" do not use the belt. If the answer is "Yes" ask user to unbuckle the belt. If user is able to easily unbuckle the belt the belt will not be considered a restraint and the belt can be used with the resident or patient. If the user has any difficulty unbuckling the belt, the belt will be considered a restraint and cannot be used. See CMS definition of restraint below.

IMPORTANT: Product is not intended to be used as a restraint. Secure[®] Quick-Releasing Seat Belts help reduce the risk of forward sliding and act as a reminder for patients to ask for assistance with ambulation. Please note that Secure[®] Quick-Releasing Seat Belts are not considered to be restraints as long as the individual is capable of releasing the closures themselves.*

*The CMS definition of a restraint, reference 42 CFR section 483.13 (e) states "SR (Self-Releasing) belts are not considered restraints if patients are capable of releasing the belt themselves."

WARRANTY: Personal Safety Corporation warrants this product to be free from factory defects in materials and workmanship for a period of one year from the date of purchase. Warranty is void if this product is misused, abused or damaged. No warranty or responsibility of use expressed or implied.

CAUTION!

**This product is not a restraint. It will not prevent a resident from getting out of their bed or chair.
This product is designed as a monitor, alerting the staff when the resident has risen from their bed or chair.
This product is to be used in conjunction with a total fall prevention program**



*Please see our complete line of Secure[®] fall/wandering prevention & patient safety products
at www.SecureSafetySolutions.com*

Highest Quality...Lowest Prices...Guaranteed!

Personal Safety Corporation • Hiawatha, Iowa 52233 • 800.373.2873