

CONVEX POMMEL CUSHION With SAFETY STRAPS

Size: 18" W x 16" L x 3.5" H

Model # SCPC-1 • Made in China • One Year Warranty

The Secure[®] Convex Pommel Cushion is designed to fill the concave gap naturally created when a resident or patient is seated in a soft vinyl sling-style wheelchair seat. The convex bottom of the cushion will fill the gap providing a flat seating surface to stabilize seating and assist with proper posture and relief of sacral pressure.

The Convex Pommel Cushion is also designed to keep hips & knees aligned to prevent hip dislocation post-hip fracture. The ultra soft high-density foam provides ultimate resident seating comfort.

NOTE: Meets California #117 Flame Retardant Standards

UNIQUE FEATURES AND BENEFITS

- **2.75" pommel will provide improved male seating comfort versus competitive 4" pommels**
- Built-In Pommel Will Minimize Resident Sliding Forward
- Can Be Used On Sling-Style Vinyl Wheelchair Seats Without Need For Additional Foundation
- High-Density Foam For Resident Comfort
- Moisture & Mildew-Resistant Vinyl
- Slip-Resistant Bottom Prevents Movement In Chair
- Safety Straps To Secure Cushion Placement
- Outer Cover Can Be Unzipped Quickly For Cleaning
- Ultra Soft Vinyl
- Easy To Clean
- Latex Free



Photo A



Photo B

HOW TO USE: Separate yellow Velcro[®] safety straps attached to the back of convex cushion. The end with the safety straps will be facing towards the back of chair. Position cushion onto wheelchair seat with slip-resistant surface facing down. (See Photo A). Slide safety straps through opening between seat back and wheelchair frame on both sides of chair. Affix Velcro[®] straps together to secure convex cushion in place. (See Photo B). Carefully assist resident into wheelchair.

CLEANING INSTRUCTIONS: Wipe clean with a damp cloth using mild soap or liquid disinfectant as needed. **DO NOT** use phenol and benzyl based disinfectants. **MUST** rinse with water after cleaning to remove any residual chemicals. Always make certain cushion is completely dry before use.

ONE YEAR WARRANTY: Personal Safety Corporation warrants this product to be free from factory defects and workmanship for a period of one year from date of purchase. Void if product's outer covering is misused, abused, cut or punctured. No warranty or responsibility to use expressed or implied.

CAUTION!

The Secure[®] Convex Pommel Cushion is not intended as a restraint. Use the pommel cushion only with residents/patients who can easily get off the pommel cushion and exit wheelchair on their own without staff member assistance. Please properly position the Secure[®] Convex Pommel Cushion according to the instructions above with each use to provide maximum support and comfort for the resident.

Please see our complete line of Secure[®] fall & wound care management products at www.PadAlarm.com.



Secure[®] Wedge Seat Cushion



Secure[®] Lap Cushion



Secure[®] Deluxe Arm Support



Please see our complete line of Secure[®] fall/wandering prevention & patient safety products at www.SecureSafetySolutions.com

Highest Quality...Lowest Prices...Guaranteed!

Personal Safety Corporation • Hiawatha, Iowa 52233 • 800.373.2873