

CONVEX SEAT SUPPORT WITH SAFETY STRAPS

Size: 18" W x 16" L x 1.75" H

Model # SCSS-1 • Made in China • One Year Warranty

The Secure[®] Convex Seat Support is designed to fill the concave gap naturally created when a resident or patient is seated in a soft vinyl sling-style wheelchair seat. The convex bottom of the seat support will fill the concave gap providing a flat, firm surface to support flat bottom wheelchair cushions.



PHOTO A



PHOTO B



PHOTO C

The seat support helps provide a solid surface to eliminate uneven weight distribution. The seat support will also assist with proper posture & comfort of your patient or resident.

NOTE: Meets California #117 Flame Retardant Standards

UNIQUE FEATURES AND BENEFITS

- Must be used as a support for other flat bottom wheelchair cushions. **DO NOT USE AS CUSHION!**
- Moisture & Mildew-Resistant Vinyl
- Slip-Resistant Bottom Prevents Movement In Chair
- Safety Straps To Secure Seat Support Placement
- Ultra Soft Vinyl
- Easy To Clean
- Latex Free

HOW TO USE: Separate yellow Velcro[®] safety straps attached to the end of the seat support. Position seat support onto wheelchair seat with slip-resistant surface facing down. Align the front edge of the seat support with the front edge of the wheelchair seat. The end with yellow safety straps will be facing towards the back of chair. (See Photo A). Place desired wheelchair cushion on top of seat support with slip-resistant side of cushion against top of seat support. Slide both sets of safety straps through opening between seat back and wheelchair frame on both sides of chair. (See Photo B) Affix Velcro[®] straps together to secure seat support in place. (See Photo C). Carefully assist resident into wheelchair.

CLEANING INSTRUCTIONS: Wipe clean with a damp cloth using mild soap or liquid disinfectant as needed. **DO NOT** use phenol and benzyl disinfectants. **MUST** rinse with water after cleaning to remove any residual chemicals. Always make certain seat support is completely dry before use.

Personal Safety Corporation warrants this product to be free from factory defects and workmanship for a period of one year from date of purchase. Void if product's outer covering is misused, abused, cut or punctured. No warranty or responsibility to use expressed or implied.

CAUTION!

This product is not a restraint. It will not prevent a resident from getting out of a wheelchair. Please properly position the Secure[®] Wedge Seat Cushion according to the instructions above with each use to provide maximum support and comfort for the resident.

Please see our complete line of Secure[®] fall & wound care management products at www.PadAlarm.com.



Secure[®] Convex Pommel Cushion



Secure[®] Lap Cushion



Secure[®] Deluxe Arm Support



Please see our complete line of Secure[®] fall/wandering prevention & patient safety products at www.SecureSafetySolutions.com

Highest Quality...Lowest Prices...Guaranteed!

Personal Safety Corporation • Hiawatha, Iowa 52233 • 800.373.2873