

Your safety is our business®

R€° DELUXE ARM SUPPORT

Model # SDAS-1 • Made in China • One Year Warranty









Photo B

Photo C Photo D

The Secure® Deluxe Arm support has been designed for resident support and comfort. Please note this cushion can be used on either right or left arm rest. However, we recommend each arm rest be covered with a Secure Deluxe Arm Support for ultimate support and comfort. One per package.

NOTE: Meets California #117 Flame Retardant Standards

UNIQUE FEATURES AND BENEFITS

- High-Density Foam For Resident Comfort
- Moisture & Mildew-Resistant Vinyl
- Ultra Soft Vinyl
- Easy To Clean
- Latex Free

Photo A

· Easy To Use

HOW TO USE

Position the Secure® Deluxe Arm Support over the wheelchair arm rest with the long side of cushion facing the inside of wheelchair to help support the resident. (See Photo A). Apply downward pressure until firmly in place. (See Photos B & C). Repeat same placement procedure if using two arm supports. Carefully assist resident into wheelchair. (See Photo D).

CLEANING INSTRUCTIONS: Wipe clean with a damp cloth using mild soap or liquid disinfectant as needed.

ONE YEAR WARRANTY: Personal Safety Corporation warrants this product to be free from factory defects and workmanship for a period of one year from date of purchase. Void if product's outer covering is misused, abused, cut or punctured. No warranty or responsibility to use expressed or implied.

CAUTION!

This product is not a restraint. It will not prevent a resident from getting out of a wheelchair. Please properly position the Secure® Deluxe Arm Support according to the instructions above with each use to provide maximum support and comfort for the resident.

Please see our complete line of Secure® fall & wound care management products at www.PadAlarm.com.









Secure® Convex Pommel Cushion

Secure® Lap Cushion



Please see our complete line of Secure® fall/wandering prevention & patient safety products at www.SecureSafetySolutions.com

Highest Quality...Lowest Prices...Guaranteed!

Personal Safety Corporation • Hiawatha, lowa 52233 • 800.373.2873