

EASY-RELEASE LAP CUSHION

Size: 20" x 4"

Model # SLC-1 • Made in China • One Year Warranty



Photo A



Photo B



Photo C



Photo D

The Secure[®] Easy-Release Lap Cushion was designed to provide anterior posture support while safely securing the resident in a seated position. **This product is not intended to be used as a restraint or to limit resident movement.** If resident is able to remove lap cushion without assistance, the lap cushion will not be considered a restraint under OBRA Regulations.

When a resident wants to stand up, they can remove the lap cushion by lifting up on the yellow handle at either end of the cushion completely separating the yellow straps from the cushion. **NOTE: Only one strap is needed to be released for resident to exit wheelchair.** The lap cushion does not hinder the resident from getting up and being ambulatory. The Secure[®] Easy-Release Lap Cushion also provides a convenient elbow rest or a work surface.

NOTE: Meets California #117 Flame Retardant Standards

UNIQUE FEATURES AND BENEFITS

- Easy-Release Velcro Straps
- Provides Resident An Elbow Rest & Work Surface
- Fits 18" – 22" Wide Wheelchairs or Geri-Chairs
- 4" Medium-Density Foam For Resident Comfort
- Moisture & Mildew-Resistant Vinyl
- Ultra Soft Vinyl
- Easy To Clean
- Latex Free

A. HOW TO INSTALL LAP CUSHION TO WHEELCHAIR

NOTE: After each installation resident must be checked to ensure he/she cannot slide down under lap cushion. If resident can slide under cushion do not use.

1. Lock the wheelchair brakes and turn up foot rests.
2. Carefully assist resident into wheelchair. Be sure resident is seated as far back in the wheelchair as possible.
3. Remove both easy-release straps by lifting up on the yellow handles and completely separating straps from the lap cushion.
4. Position the lap cushion between the arm rests. (See Photo A).

NOTE: You may need to compress the sides of the lap cushion for proper positioning or placement.

5. Insert the yellow straps through wheelchair frame in the front of the wheelchair under the arm rest. (See Photo B).
6. Secure the yellow straps by first pulling them away from cushion and then slowly and firmly press yellow straps to Velcro[®] sewn on cushion.
7. Adjust the lap cushion as needed for resident comfort and use. Get confirmation from resident that cushion is in best position for his/her use. (See Photo C).

NOTE: The resident should always be comfortable and breathing should not be restricted. An open hand should freely be able to pass between front of resident and cushion.

B. HOW TO REMOVE LAP CUSHION FROM WHEELCHAIR

1. To completely remove the lap cushion from wheelchair simply pull up on both yellow handles of straps, separate straps from cushion and pull straps through slots on both sides of wheelchair. (See Photo D).
2. Carefully assist resident in getting out of wheelchair.

Continued



Please see our complete line of Secure[®] fall/wandering prevention & patient safety products at www.SecureSafetySolutions.com

Highest Quality...Lowest Prices...Guaranteed!

Personal Safety Corporation • Hiawatha, Iowa 52233 • 800.373.2873

EASY-RELEASE LAP CUSHION *(Continued)*

Size: 20" x 4"

Model # SLC-1 • Made in China

CAUTIONS:

1. This product must be tested by asking the resident to remove their lap cushion each time it is used to be sure they can remove lap cushion on their own.
2. If the resident cannot easily self-release this lap cushion on his/her own, it would be considered a restraint and therefore would need to be prescribed by a physician.

NOTE: Residents who cannot ambulate safely on their own or who are a fall risk should not use this product alone. Please see our catalog for additional safety products to assist with fall management.

CLEANING INSTRUCTIONS: Wipe clean with a damp cloth using mild soap or liquid disinfectant as needed.

ONE YEAR WARRANTY: Personal Safety Corporation warrants this product to be free from factory defects and workmanship for a period of one year from date of purchase. Void if product's outer covering is misused, abused, cut or punctured. No warranty or responsibility to use expressed or implied.

WARNING!

This product is not a restraint. It will not prevent a resident from getting out of their chair. If the resident is not able to easily self-release this product, it would be considered a restraint. Lap cushions should always feel comfortable and not restrict breathing. Discontinue use of this product immediately if resident becomes uncomfortable or agitated during use. Please properly position the Secure[®] Easy-Release Lap Cushion according to the instruction above with each use to ensure resident safety and comfort. This product is to be used in conjunction with a total fall prevention program.

Please see our complete line of Secure[®] fall & wound care management products at www.PadAlarm.com.



Secure[®] Wedge Seat Cushion



Secure[®] Wedge Pommel Cushion



Secure[®] Deluxe Arm Support



Secure[®] Beveled Edge Safety Mat



Secure[®] Bedside Floor Mat



Please see our complete line of Secure[®] fall/wandering prevention & patient safety products at www.SecureSafetySolutions.com

Highest Quality...Lowest Prices...Guaranteed!

Personal Safety Corporation • Hiawatha, Iowa 52233 • 800.373.2873