

# Loud-Mate<sup>®</sup> Caregiver Alarm

**120+ Decibel Resident Safety Alarm For Use In Rooms Farthest From the Nurse's Station**

Model # SLM-1 • Made in PRC

## Test/Flashlight Button

1. Press button to test. Alarm will sound and light will flash for 3 seconds then automatically shut off to ensure proper functioning.
2. Press & continue to hold button. Light will illuminate and function as a flashlight until you release button.

Locking Clip



Flashing alert light instantly identifies person in need...day or night.

**One Year Replacement Warrantly**

Size: 2-1/4" x 2-3/4" x 1"

Created by and for long-term care professionals. Used in thousands of concerned care facilities across the U.S.


- 120 decibel alarm for use in rooms farthest from nurse's station
- Flashing alert light to instantly identify person in need
- 4' adjustable cord with locking clip to secure to clothing
- 9-volt battery included
- Side switch to test battery and ensure alarm functions properly
- Velcro<sup>®</sup> attachment

**TO USE:** If using Velcro<sup>®</sup> strip on back of alarm, remove paper backing and affix alarm to bed, chair, wheelchair or nightstand out of reach of resident. If using one of the Alert Mate<sup>®</sup> Holders, AMH-1 or AMH-2 (sold separately), insert alarm into holder and attach to bed, chair, wheelchair or nightstand out of reach of resident. **CAUTION: Do not affix alarm to a PVC rail. The strap may permanently stain the rail.** Insert pin at end of cord in either front or top pin receptacle of Secure<sup>®</sup> Loud-Mate<sup>®</sup> Alarm. Adjust cord to desired length. Attach cord to resident's clothing with locking clip so that the clip is not accessible to the resident. Insert pin at end of cord in top pin receptacle of alarm. Either pre-test with a staff member or have the resident stand up to ensure pin will release from alarm when the user stands up. If resident moves beyond desired range, pin will come out and alarm will alert staff. Test alarm each time it is used to make sure it functions properly.

**ASK YOUR ACCOUNT SERVICE MANAGER ABOUT THE ALERT-MATE<sup>®</sup> REMOVEABLE ALARM HOLDER.**

## TO INSTALL BATTERY

1. Loosen the two screws on back of alarm with a small Phillips or slotted screwdriver
2. Remove back of alarm
3. Install 9-volt battery as indicated
4. Replace back panel and tighten screws

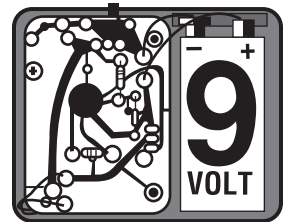

**WARRANTY:** Personal Safety Corporation warrants this product to be free from factory defects in materials and workmanship for a period of one year from date of purchase. Warranty is void if this product is misused, abused or damaged. No warranty or responsibility of use expressed or implied.

## CAUTION!

**This product is not a restraint, therefore will not prevent a resident from getting out of their bed or chair. This product is designed as a monitor, alerting the staff when the resident has risen from their bed or chair. This product is to be used in conjunction with a total fall prevention program.**



Please see our complete line of Secure<sup>®</sup> fall/wandering prevention & patient safety products at [www.SecureSafetySolutions.com](http://www.SecureSafetySolutions.com)

**Highest Quality...Lowest Prices...Guaranteed!**

**Personal Safety Corporation • Hiawatha, Iowa 52233 • 800.373.2873**