

PADDED GAIT BELT

WITH 7 HAND GRIPS & AUTO STYLE BUCKLE, 70" L X 4" W

Fits Waist Sizes 28"-60"

Model No. STWB-70A • Made in PRC

Instructions

The Secure[®] Padded Gait Belt helps prevent caregiver back injuries and assist in safe patient transfers and ambulation. The Padded Gait Belt is wider than traditional gait belts for greater contact area with the patient. Fits waist sizes 28"-60". Each belt comes with multiple vertical and horizontal hand grips that allow the caregiver to choose the best leverage point.

The Secure[®] Padded Gait Belt is to be used as part of a total fall management program. Please note that the belt should only be used as authorized by the results of the resident's risk assessment. The Secure[®] Padded Gait Belt is to be used as an effective aid when lifting, walking with or transferring a resident or patient. **AT NO TIME IS THE BELT TO BE USED AS A RESTRAINT TO SECURE A RESIDENT TO A BED, CHAIR OR A STATIONARY OBJECT OF ANY KIND.**

VERY IMPORTANT! ASSESS WHETHER THE RESIDENT'S SIZE AND/OR WEIGHT WILL REQUIRE MORE THAN ONE PERSON TO SAFELY LIFT, WALK WITH OR TRANSFER RESIDENT. WHEN IN DOUBT, TWO CAREGIVERS SHOULD ALWAYS BE USED FOR RESIDENT SAFETY.

1. Position the belt around the resident's waist.
DO NOT POSITION BELT AROUND RESIDENT'S CHEST.
2. Adjust belt length to fit securely around the waist. Make sure the belt is snug, but not uncomfortable. You should be able to slide your open hand (flat) between the belt and the patient.
3. Secure the quick-release buckle together by inserting male side of buckle into female side of buckle. The belt is now positioned for use.
4. Use both hands to firmly grip the belt using the attached hand grips located along the back of the belt. If two caregivers are required, position one caregiver on the right side and one caregiver on the left side of the resident. You are now ready to safely lift, walk with or transfer the resident or patient.

Cleaning Instructions

Hand wash belt section in warm water, line dry. Do not wash or immerse the buckle in water. Buckle may be wiped with disinfectant cleaner if needed.

One Year Limited Warranty

Personal Safety Corporation warrants this product to be free from factory defects and workmanship for a period of one year from date of first use. Void if this product has been misused, abused or cut. No warranty or responsibility of use expressed or implied.

WARNING:

NEVER use a Gait Belt if there is any compromise of safety or indication of inappropriateness to the patient or caregiver. **ALWAYS** test to make sure the belt is applied securely and will not loosen when transferring patient or resident. **INSPECT** before each use by checking for broken stitches, torn material, or cracked buckles.



Please see our complete line of Secure[®] fall/wandering prevention & patient safety products at www.SecureSafetySolutions.com

Highest Quality...Lowest Prices...Guaranteed!

Personal Safety Corporation • Hiawatha, Iowa 52233 • 800.373.2873