


EZ RELEASE SOFT WHEELCHAIR SEAT BELT – BLACK/GRAY

Model #: SWSB-1B • Made in PRC

The Secure[®] EZ Release Soft Wheelchair Seat Belts are intended to act as “gentle” reminders for patients to ask for assistance when leaving the wheelchair. The belts also lessen the likelihood of falling forward out of the wheelchair.



kickspur and the strap from the patient’s left side over the right kickspur. (See illustration C) Adjust the sliders to take up any slack in the straps. The belt should fit snugly and comfortably around the patient’s lower torso.

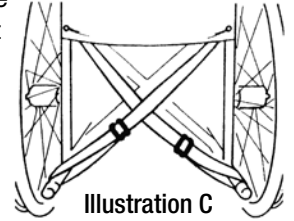

CAUTION

These devices are not restraints! Belts are intended to be used only on patients who are able to remove the belt without assistance. The determination as to the appropriateness of the EZ Release Soft Wheelchair Seat Belt for a specific patient is the sole responsibility of the facility and its professional and medical staff.

TO CLOSE AND OPEN QUICK-RELEASE BUCKLE

Insert male buckle end into female end. The buckle is secure when you hear a click and see the securing tabs on the male end protruding through the openings on both sides of the female end. To release, simultaneously press the tabs on both sides of the buckle.

IMPORTANT!

Periodically check belt to make certain that the closure is properly secured. It is especially important to check the closure if the patient has returned to the wheelchair and reapplied the belt without assistance from a caregiver.

INSTRUCTIONS FOR USE

1. Before seating your patient, lock both wheelchair brakes and place the footrests in the out-of-the-way position. Assist patient into the wheelchair and position him as far back on the wheelchair seat as possible. Reposition the footrests and place patient’s feet on them.
2. With the closure secured, place the belt at the patient’s waist and bring the straps directly behind the patient. Thread the straps through the space between wheelchair seat and backrest.
3. Behind the wheelchair, criss-cross the straps and loop the strap from the patient’s right side over the left

LAUNDERING

This device can be laundered and dried at temperatures up to 180OF (82OC). Bleaching is not recommended.

SIX MONTH LIMITED WARRANTY

Personal Safety Corporation warrants this product to be free from factory defects and workmanship for a period of six months from date of purchase. Warranty is void if this product is misused, abused or damaged. No warranty or responsibility of use expressed or implied.

Visit www.PadAlarm.com for more Fall/Wandering Management Products



Secure[®] Non-Alarming Quick-Release Wheelchair Seat Belt



Secure[®] Easy-Release Wheelchair Lap Cushion



Secure[®] Transfer & Walking Belt w/EZ Release Plastic Buckle



Please see our complete line of Secure[®] fall/wandering prevention & patient safety products at www.SecureSafetySolutions.com

Highest Quality...Lowest Prices...Guaranteed!

Personal Safety Corporation • Hiawatha, Iowa 52233 • 800.373.2873