

SecureSleeves™ For Legs

Small/Medium • Large/X-Large • XX-Large

**Model #s - LEGS-S/M Brown, LEGS-L/XL Brown, LEGS-XXL Brown,
LEGS-S/M Ivory, LEGS-L/XL Ivory, LEGS-XXL Ivory**




Step 1

The super soft SecureSleeves™ For Legs are designed to protect sensitive skin from tears and abrasions while providing ultimate wearer comfort. Made from a proprietary blend of Chitosan rayon, nylon, cotton and Spandex, SecureSleeves™ For Legs anti-microbial properties protect against odor causing bacteria often-accompanying deep

wound and skin problems. In addition the anti-microbial properties of Chitosan rayon protects against the growth of bacteria, molds and fungi. SecureSleeves™ can also be used to cover I.V. and wound sites. Machine washable and Latex-free.



Step 2

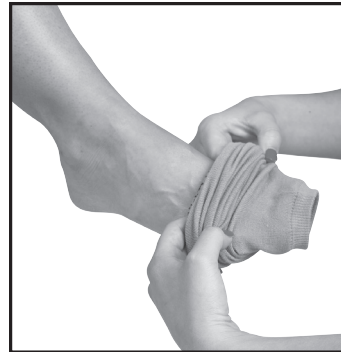

Step 3



Step 4

INSTRUCTIONS FOR USE

1. Hold the SecureSleeve™ between your two hands at the top Secure logo band of the sleeve. The heel section of the sleeve will be facing away from you. See Step 1 photo above.
2. Using your fingers and thumbs begin to pull the sleeve upward in an accordion-like fashion until you can no longer gather the material. See Step 2 photo above.
3. Carefully slip the SecureSleeve™ over the patient's/resident's foot and continue moving the sleeve upward until the patient's/resident's heel fits into the heel section of the sleeve. See Step 3 photo above.
4. Slowly and carefully begin pushing the sleeve upwards releasing the accordion pleats until the sleeve is fully extended on the patient's/resident's leg. Adjust the sleeve on the foot and at the top of the leg to achieve maximum patient/resident comfort. See Step 4 photo above.

Caution: Please make certain the bottom and top of the sleeve are not binding around the patient's/resident's foot or leg. Check for a comfortable, non-binding fit. **Please note:** **Not recommended for very small legs or very large legs.**

CAUTION

Conduct frequent and careful visual checks of the patient's/resident's skin to observe changing conditions of existing skin conditions being treated and take appropriate treatment action for any new skin problems. Follow facility policy with respect to replacing or laundering patient's/resident's SecureSleeve™.

CLEANING INSTRUCTIONS

Machine wash at lukewarm (110°F, 38°C) and dry at low setting. Hang dry for longer sleeve life.



Please see our complete line of Secure[®] fall/wandering prevention & patient safety products at www.SecureSafetySolutions.com

Highest Quality...Lowest Prices...Guaranteed!

Personal Safety Corporation • Hiawatha, Iowa 52233 • 800.373.2873