



ADJUSTABLE FALL MANAGEMENT BED RAIL WITH FLOOR SUPPORT


Model #: EZBR-1W • Made in Taiwan

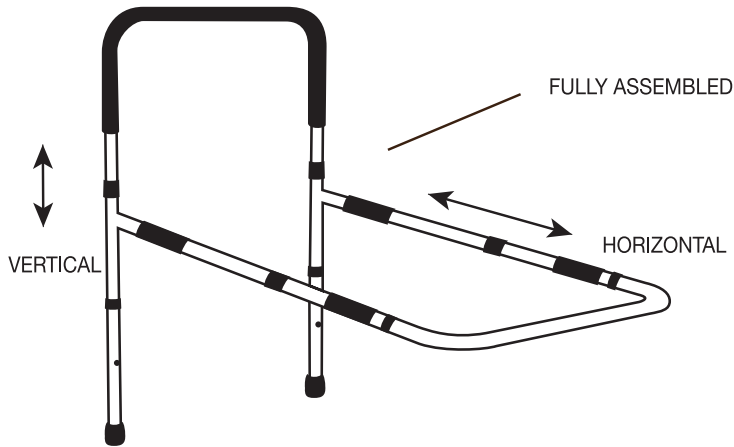

See assembly illustration to the right for the main 2 sections (Vertical & Horizontal).

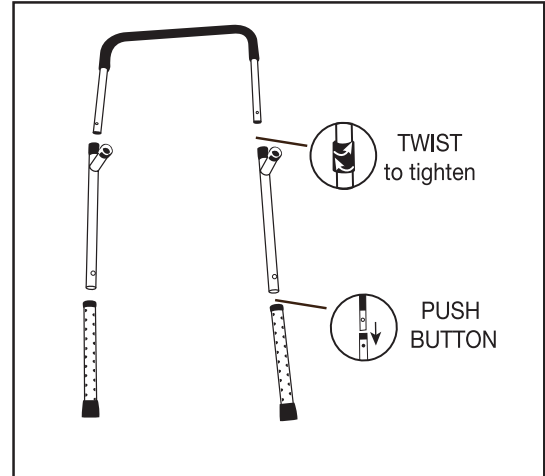


VERTICAL Parts

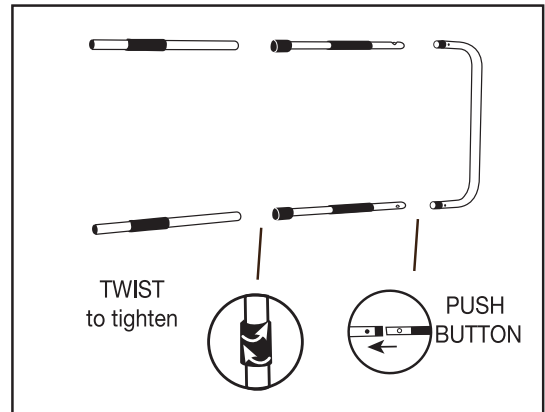


HORIZONTAL Parts

ASSEMBLY INSTRUCTIONS

CAUTION: Please inspect all parts carefully for cracks/breaks prior to use.

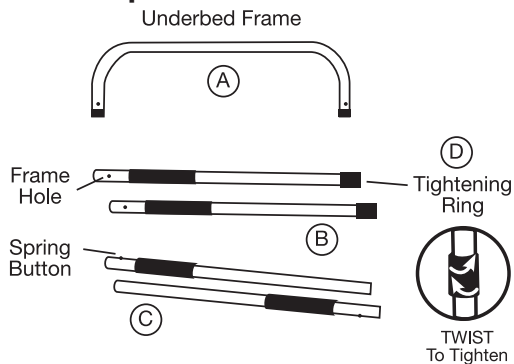


Support Rail Legs

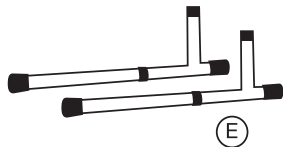

1. Assemble underbed frame parts (A), (B), and (C) with spring buttons. For positive assembly, be sure all spring buttons pop through the frame hole and you hear a "click". Tighten parts (B) and (C) together by twisting plastic ring (D).

2. Insert frames (B) into support rail legs (E).

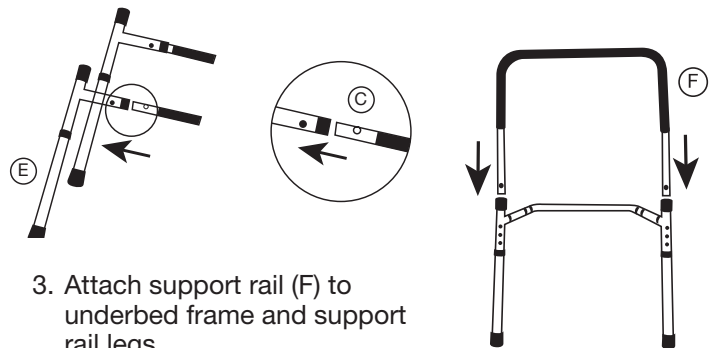

3. Attach support rail (F) to underbed frame and support rail legs.
4. Maximum weight limit: 300 lbs.


Continued

ADJUSTABLE FALL MANAGEMENT BED RAIL

Continued

ADJUSTMENT INSTRUCTIONS


1. To adjust depth of frame to go under mattress:
2. To adjust leg height:
3. To adjust handle height:

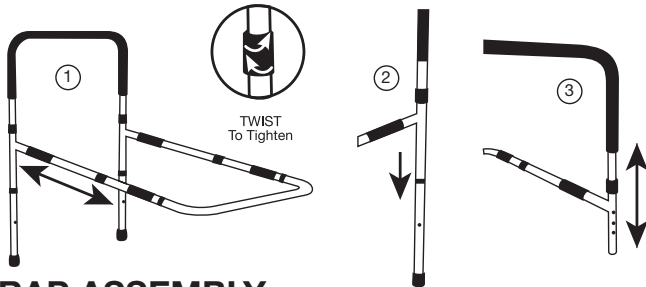

STRAP ASSEMBLY

Safety strap provides extra stability, especially for hospital and twin beds where the mattresses are lighter and smaller. Strap is designed to fit all mattress sizes.

1. Insert bed rail under mattress.
2. Go to the opposite side of the bed and lift mattress until bottom stability bar of bed rail is visible.
3. Place strap around bed rail bottom stability bar and around bed frame bar directly below. Insert buckle ends of strap and adjust until strap is secured with a snug fit.

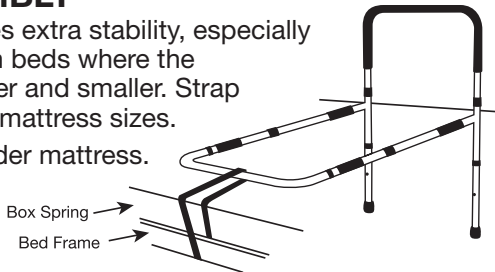


USER INSTRUCTIONS

Side rails are for the purpose of aiding in turning and repositioning within the bed and are not intended to be used for restraint purposes.

CAUTION: Be sure push buttons are aligned equally. Push buttons must fully protrude through the adjustment hole to ensure a positive lock.

This product can be used as an aid in balance and support when getting in and out of the bed. This product is NOT designed to support the total weight of the user.

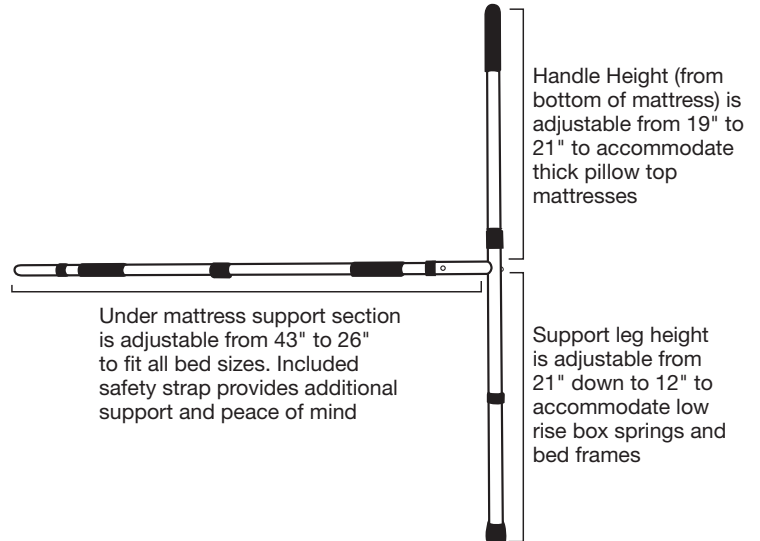

CARE INSTRUCTIONS

Periodically check that all push buttons are securely engaged and tightening rings are tight.

CAUTION: POSSIBLE INJURY. Make sure side rails are secured properly before use.

RISK OF SERIOUS INJURY. Individuals with physical limitations who cannot prevent themselves from rolling or climbing out of the bed may require other means of safe positioning.

Side rails do not fall within any weight limitations and may be damaged if excessive pressure is placed on them.

One Year Limited Warranty

Personal Safety Corporation warrants this product to be free from factory defects and workmanship for a period of one year from date of purchase. Void if this product is misused, abused altered, modified or used for purposes other than as an adult bed rail. No warranty of use expressed or implied.

CAUTION! This product is not a restraint. It will not prevent a resident from getting out of their bed. This product is to be used in conjunction with a total fall prevention program.



Please see our complete line of Secure[®] fall/wandering prevention & patient safety products at www.SecureSafetySolutions.com

Highest Quality...Lowest Prices...Guaranteed!

Personal Safety Corporation • Hiawatha, Iowa 52233 • 800.373.2873